



## Directions for Use – Pre-Stained Morphology Slides

General chamber preparation for hematology testing

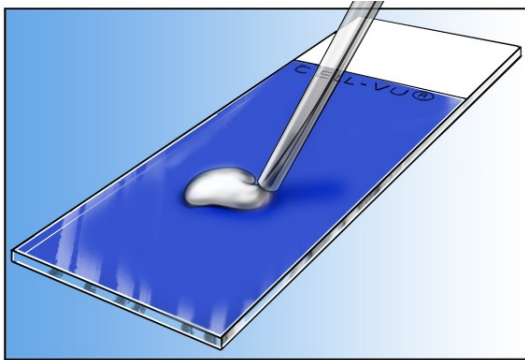


**Step 1:** Apply a small drop (approximately ~2-4 microliters) of specimen on the center of the slide.

**Warning:** EDTA is the recommended anticoagulant for venous blood. Oxalae, flouride and heparinized blood are NOT suitable.

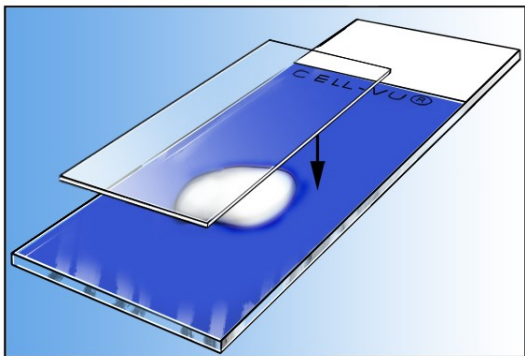

**Step 2:** Place the cover glass (24 x 24 mm or 24 x 40 mm) on top of the slide, exerting slight pressure on the coverglass.

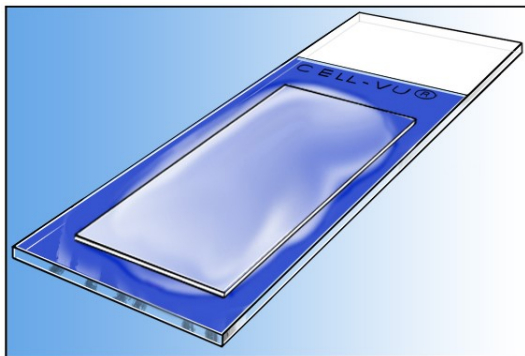

**Step 3:** Allow the slide to stand at room temperature for 15 minutes before microscopic observation is made. The slides are used for evaluating sperm head/acrosome size, differentiation of white blood cells in peripheral blood and cytology of body fluids.

**Expected Findings:**

These slides are for evaluating sperm head/acrosome size, differentiation of white blood cells in peripheral blood and cytology of body fluids.

**Order Code:**

---

VU-DRM 900

Cell-Vu Pre-Stained Morphology Slides

75/box

---