## DEUTSCH

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## ENGLISH

VSPERM MORPHO SLIDE is a ready-to-use pre-stained slide for staining human sperm.

### COMPOSITION

- New Methylene Blue
- Cresyl Violet Acetate

### CONTRAINDICATION

No known contraindication.

### INSTRUCTIONS FOR USE

VSPERM MORPHO SLIDE should be used only for its intended purpose by qualified and trained persons.

- ① Pipette 10 µL of the well-mixed, liquefied semen sample directly onto the marked centre of the labelled slide. Warning: For spermatozoa densities of less than 1 million/mL, centrifuge the semen for 10 minutes at 300 x g and place a 10 µL drop of the pellet onto the slide. For spermatozoa densities greater than 100 million/mL use a smaller volume.
- ② Place a cover slip on the specimen immediately.
- ③ Allow the prepared slide to settle for at least 10 - 25 minutes at room temperature before evaluating. Examination can be started as soon as the spermatozoa are immotile.
- ④ Examine the slide under oil immersion at 1000 x magnification.

### STORAGE INSTRUCTIONS AND STABILITY

- Leave product in its original packaging.
- Do not use after expiry date.
- Keep away from sunlight.

### WARNING AND PRECAUTIONS

Handle all samples as if capable of transmitting HIV or hepatitis. Always wear protective clothing when handling samples. Only for intended use. Ensure a correct disposal.

## DE / EN

VSPERM MORPHO SLIDE

Stück pro VE / Piece per unit

REF

10 Objektträger / Slides

V-SMS-01

Technischer Support  
Technical support

VITROMED GmbH  
Raiffeisenstr. 15a  
40764 Langenfeld  
Germany

+ 49 2173 . 200 41 30  
support@vitromed.com  
www.vitromed.com

Stand der Informationen  
Date of information

23.01.2020