


# Operation guide for Staining kit for spermatozoan morphology (Diff-Quik rapid staining)

Ref# BRED-015

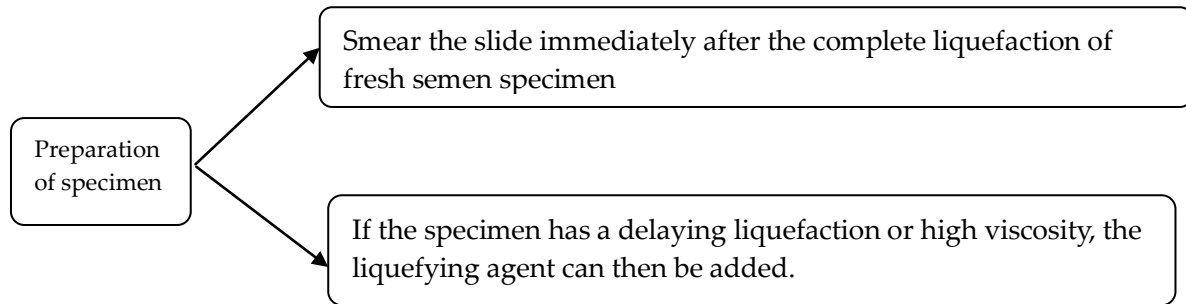

## Test steps:

1. Make a smear  $\xrightarrow{\text{Air dry}}$  Add to fixation solution  $\xrightarrow{15 \text{ seconds}}$  Add to dye solution A  $\xrightarrow{10 \text{ seconds}}$  Add to dye solution B  $\xrightarrow{15 \text{ seconds}}$  Rinse by running water 10-15 times
2. Air dry  $\longrightarrow$  Observe at least 200 spermatozoa using general optical microscope with 100 $\times$  field lens

## Precautions:

1. Only wash the slide by running water at the last procedure.
2. The staining results for some specimens may be not good after standard procedure, we could prolong the staining time of dye solution A and dye solution B.